

Files Glide In. Debris Lifts Out.



Softens calcification for easier access

Gel-like consistency for long-lasting lubrication

18% EDTA for excellent debris-lifting effervescing action and removal of smear layer

Disposable, single-dose syringe tips eliminate cross-contamination

Success starts with Glyde File Prep® root canal conditioner.

Cleaning and shaping canals is more efficient with Glyde File Prep root canal conditioner. Its smooth, gel-like consistency clings to endodontic files, making placement in the canal easier and more convenient. And because it effervesces when it reacts with sodium hypochlorite (NaOCl); pulp tissue, dentinal shavings and debris are easily lifted away. The result – cleaner dentinal tubules and lateral canals for more successful root canal procedures.



Glyde File Prep[®] Root Canal Conditioner



Glyde File Prep root canal conditioner acts as both a lubricant and chelating agent during canal cleaning and shaping. The EDTA in Glyde File Prep gel softens calcification. It reacts with sodium hypochlorite, creating an effervescent action to lift debris coronally. The EDTA also removes the smear layer and leaves dentinal tubules and lateral canals ready for obturation.

***Get the one that clings, cleans and clears
the way for more successful root canal procedures.***

- Clings to files, making placement in the canal easier and more convenient
- Effervesces to lift debris coronally for easy removal
- Removes smear layer for improved obturation
- Eliminates cross-contamination by dispensing a clean, single dose through disposable syringe tips

Order Today.

Glyde File Prep [®] Root Canal Conditioner		
Product	Item#	Description
Syringe Kit	673021	Five 3ml syringes and 50 disposable tips
Tip Refill	673022	50 disposable tips

**DENTSPLY
MAILLEFER**
In Endodontics, Quality Has A Name.
MAILLEFER

800-924-7393 www.maillefer.dentsply.com 800-924-7389 (fax)

5100 East Skelly Drive, Suite 300, Tulsa, OK 74135